

## Como é composta uma lente?



## Você sabe o que são as marcações?

As marcações são símbolos e/ou caracteres nas lentes que identificam o material, o design e também o valor de adição, que indica o grau de presbiopia. Além dessas, contamos também com personalização das iniciais do usuário, adicionando até 4 letras!

## Todas as marcações disponíveis em uma lente Varilux®

**MARCAÇÃO SUPERIOR:**

É a identificação opcional, de até quatro iniciais escolhidas pelo usuário. Além disso, é nesse espaço em que a personalização da lente está indicada, dependendo do produto. Por exemplo, 4D para Varilux® XR Pro ou F para personalização Fit Eyecode\*.

\*Consulte a disponibilidade

**MARCAÇÃO NASAL:**

É a identificação do material e design.

**MARCAÇÃO TEMPORAL:**

É a identificação do valor de adição.

**FOG ID:**

Diferente da marcação visível contra a luz, o FOG ID aparece quando a lente encontra-se embaçada com ar quente. Indica que o produto possui o tratamento antirreflexo Crizal.



The image shows a pair of glasses with a dark, tortoiseshell frame. The left lens has several markings: "ABCD" in the upper left, "4D" below it, a small circle with "27" below that, and "Varilux" at the bottom. The right lens has a small circular marking in the center. A circular inset below the glasses shows a close-up of the "Varilux" marking on the lens. Lines connect the text boxes to the corresponding markings on the glasses.


### Lente Varilux® XR Pro no material Stylis 1.67

Essa marcação só pode ser vista contra a luz e, podem ser encontradas em ambas as lentes.  
**Não conseguiu verificar? Fale com o SAC: 4000-2595.**


DESIGNS

Clique no design abaixo para saber mais.

Varilux®  
XR Pro™




Varilux®  
XR Track™




Varilux®  
XR Track lite™




Varilux®  
XR Design™




Varilux®  
Physio®.extensee™  
track




Varilux®  
Physio®.extensee™




Varilux®  
Comfort Max




Varilux®  
Comfort




Varilux®  
Liberty™ 3.0



Varilux®  
Liberty™



Varilux®  
Digitime®.near




Varilux®  
Digitime®.mid



Varilux®  
Roadpilot



Varilux®  
Sport



Varilux®  
Sport WRAP



MATERIAIS

Orma

Airwear

P

Stylis 1.67



Stylis 1.74

74

PERSONALIZAÇÃO

Fit Eyecode F



Varilux® XR series™

Marcação

BlueUV™

Filter System

Varilux®

XR pro™

Orma	 4D
Airwear 	 P 4D
Stylis 1.67	 T 4D
Stylis 1.74	 74 4D



Varilux® XR series™

Marcação

BlueUV™

Filter System

Varilux®

XR track™

Essilor Fit Eyecode

Orma	 F
Airwear 	 P F
Stylis 1.67	 T F
Stylis 1.74	 74 F



Varilux® XR series™

Marcação

BlueUV™

Filter System

Varilux®

XR Track lite™

Orma	
Airwear 	
Stylis 1.67	
Stylis 1.74	



VOLTAR PARA DESIGNS



Varilux® XR series™

Marcação

BlueUV™

Filter System

Varilux®

XR design™

Essilor Fit Eyecode

Orma	 F	
Airwear 	 P F	 P
Stylis 1.67	 $\pi$ F	 $\pi$
Stylis 1.74	 74 F	 74



ABCD  
F  
27  
P  
VARILUX

Varilux® Physio® extensee™

Marcação



BlueUV™

Filter System

Varilux®

Physio. extensee™

Essilor Fit Eyecode

Orma	 F	
Airwear 	 P F	 P
Stylis 1.67	 $\pi$ F	 $\pi$
Stylis 1.74	 74 F	 74



ABCD  
27  
P  
VARILUX

Varilux® Physio® extensee™ track

Marcação





BlueUV™

Filter System

Varilux®

Physio. extensee™ track

Essilor Fit Eyecode

Orma	 / F
Airwear 	 / P F
Stylis 1.67	 $\pi$ F
Stylis 1.74	 74 F



ABCD  
F  
27  
P  
VARILUX

VOLTAR PARA DESIGNS



Varilux® Comfort Max

Marcação

BlueUV™  
Filter System

Varilux®  
Comfort Max

Essilor Fit EyecodeShort

Orma	⌚ ⧫ F	⌚ S	⌚ ⧫
Airwear <b>POLYPOWER™</b>	⌚ P ⧫ F	⌚ P S	⌚ P ⧫
Stylis 1.67	⌚ ⏏ ⧫ F	⌚ ⏏ S	⌚ ⏏ ⧫

Varilux® Comfort

Marcação

BlueUV™  
Filter System

Varilux®  
Comfort

Orma	⌚
Airwear <b>POLYPOWER™</b>	⌚ P

Varilux® Liberty 3.0

Marcação

BlueUV™  
Filter System

Varilux®  
Liberty™ 3.0

ShortDigital

Orma	⌚ ≡ ⧫ S	⌚ ≡ ⧫
Airwear <b>POLYPOWER™</b>	⌚ ≡ P ⧫ S	⌚ ≡ P ⧫

VOLTAR PARA DESIGNS



LINHA TRADICIONAL



Varilux® Liberty

Marcação

BlueUV™

Filter System

varilux®  
Liberty™

Orma	
Airwear POLYPOWER™	 P






Varilux® Activities™

Marcação

BlueUV™

Filter System

varilux®  
Digitime® near

Orma	
Airwear POLYPOWER™	 P
Stylis 1.67	 11






Varilux® Activities™

Marcação

BlueUV™

Filter System

varilux®  
Digitime® mid

Orma	
Airwear POLYPOWER™	 P
Stylis 1.67	 11



VOLTAR PARA DESIGNS



Varilux® Activities™

Marcação  
BlueUV™  
Filter System

Varilux®  
Sport

Airwear POLYPOWER™

 P



Varilux Sport glasses with a tortoiseshell frame. The left lens has a small circle with '27' below it. The right lens has a small circle with a P logo and 'P' to its right. The Varilux logo is at the bottom of the frame.

Varilux® Activities™

Marcação  
BlueUV™  
Filter System

Varilux®  
Sport WRAP

Airwear POLYPOWER™

WR+  P



Varilux Sport WRAP glasses with a tortoiseshell frame. The left lens has 'ABCD' and 'WR+' above a small circle with '27' below it. The right lens has a small circle with a P logo and 'P' to its right. The Varilux logo is at the bottom of the frame.

Varilux® Activities™

Marcação  
BlueUV™  
Filter System

Varilux®  
Roadpilot

Orma

 DAY & NIGHT




Varilux Roadpilot glasses with a tortoiseshell frame. The left lens has 'ABCD' above a small circle with '27' below it. The right lens has a small circle with a Day & Night logo and 'DAY & NIGHT' to its right. The Varilux logo is at the bottom of the frame.

[VOLTAR PARA DESIGNS](#)

# Gabarito

Varilux® é marca registrada da Essilor Internacional. DMK Cod.: 116281 Mar/24

\*No caso de impressão do material, as medidas em milímetros devem ser respeitadas para que o gabarito seja utilizado de maneira correta.

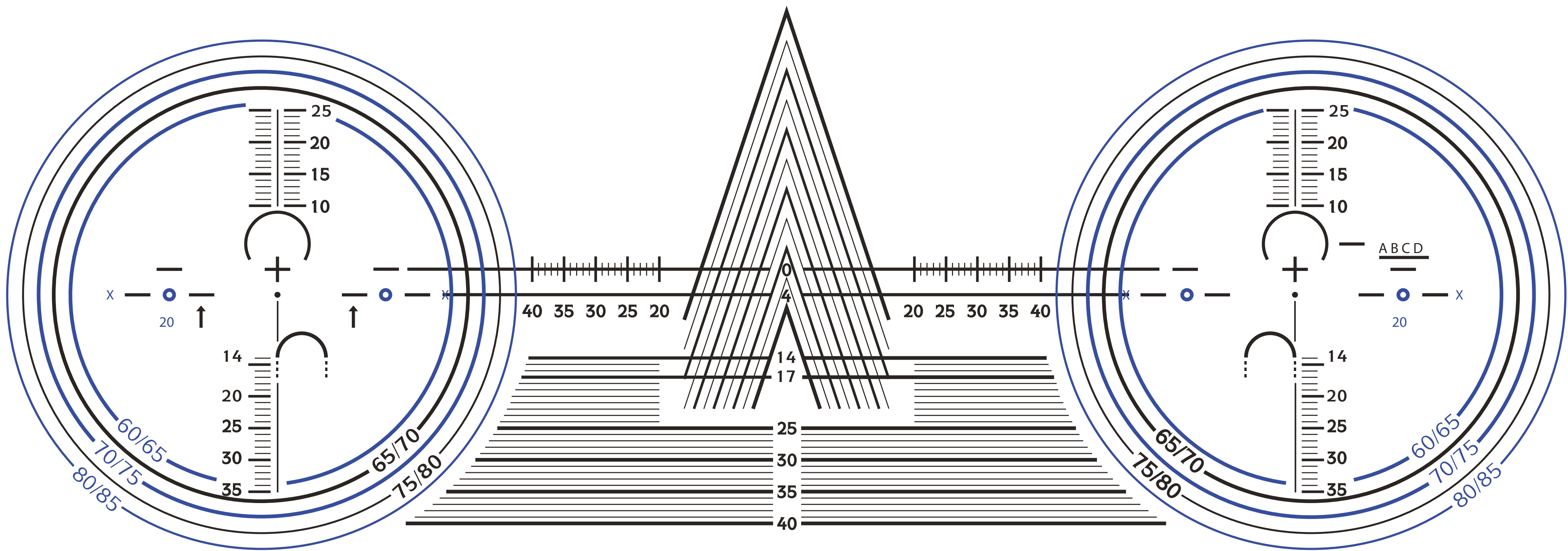

Ainda está com dúvida? Fale com o SAC 4000-2595

Varilux® XR series, Varilux® XR Pro, Varilux® XR Track, Varilux® XR Track Life, Varilux® XR Design, Varilux®, Varilux® ‘Physio Extensee Track, Varilux® Physio Extensee, Varilux® Comfort, Varilux® Liberty, Varilux® Digitme Near, Varilux® Digitme Mid, Varilux® Roadpilot, Varilux® Sport, Varilux® Sport Wrap, Essilor Fit Eyecode, Essilor® Stylis, Essilor Airwear e Orma são marcas registradas da Essilor International. Fotos meramente ilustrativas.